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The screenshot shows the DENTIXsoft 2013.2 BETA interface. The patient record for Tobias Feigel (BARMER > BAYERN, 18.05.1983 / 29 Jahre / 12100) is displayed. The 'Pläne' tab is selected. In the 'Zahnärztliche Vorgänge' section, the 'Plan.' button is highlighted with a yellow arrow. A dropdown menu is open, showing the following options: HKP, Funktionsstatus, IP, KFO, Kieferbruch, Parodontalstatus (highlighted with a yellow arrow), VDDS_Media, GammaDental, and Abrechnen. The interface also shows a 'ToDo-Liste' on the left and a 'Zahnärztliche Vorgänge' table at the bottom.

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The screenshot shows the DENTIXsoft 2013.2 BETA software interface. The window title is "Dentixsoft 2013.2 BETA Ben.: Gemeinschaftspraxis Beh.: Diana Feigel Abr. Arzt: Gemeinschaftspraxis". The main window displays a patient record for Tobias Feigel, born 18.05.1983, living at BARMER > BAYERN. The interface includes a menu bar (Datei, Abrechnung, Statistik, Extras, Einstellungen) and a toolbar with various icons. The main window is divided into several sections: "Bereits vorhandene PA-Pläne:" (Aktuell), "Befund" (Blatt 1, Blatt 2), and a dental chart. The dental chart shows a PA-sonde for tooth 16, which is crossed out with a large 'X'. The chart also shows other teeth with radiographs and a red horizontal line indicating the level of the sonde. The patient's name, address, and date of birth are visible at the top of the window. At the bottom of the window, there are buttons for "Labor", "PaOn-Sonde", "Drucken", "Übernehmen", and "Abbrechen".

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