
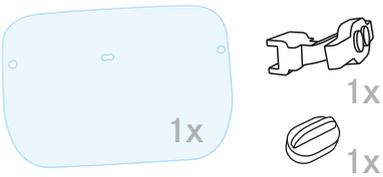
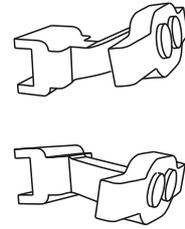
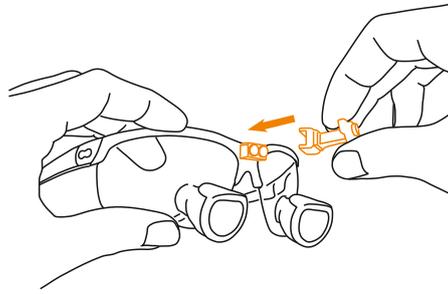
Lieferumfang  
Scope of delivery

1

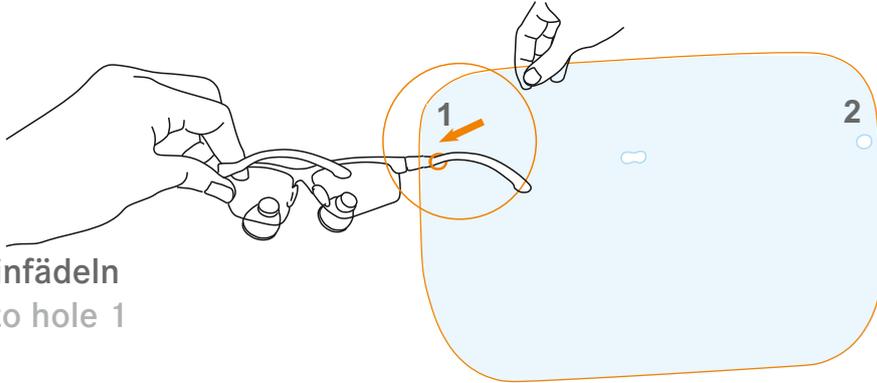
2


Loch 1 einfädeln  
Insert into hole 1

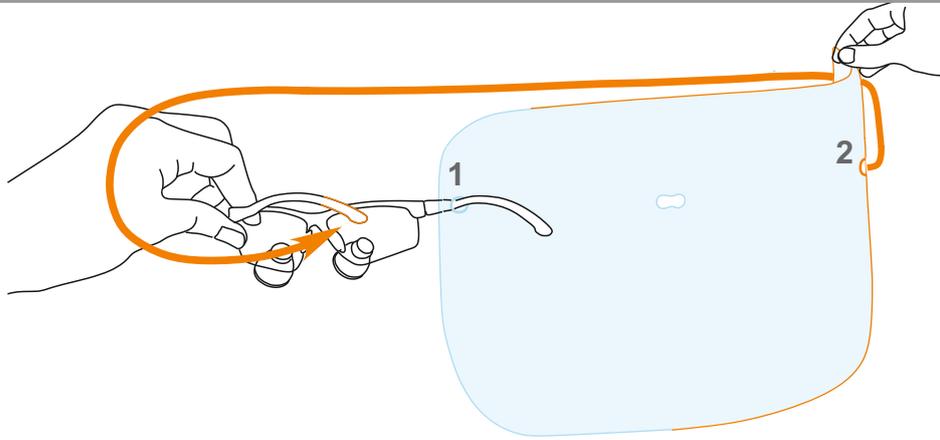3



4

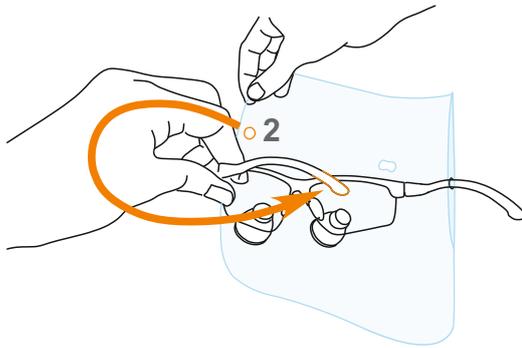
6a

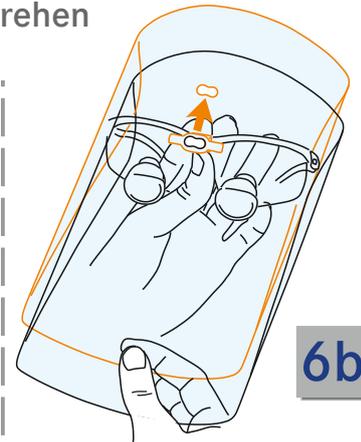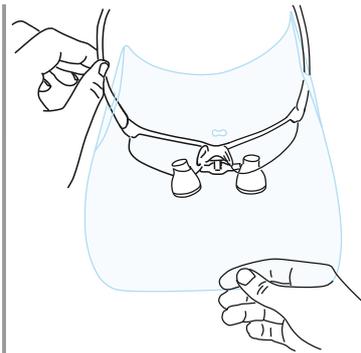Umdrehen  
Turn


6b

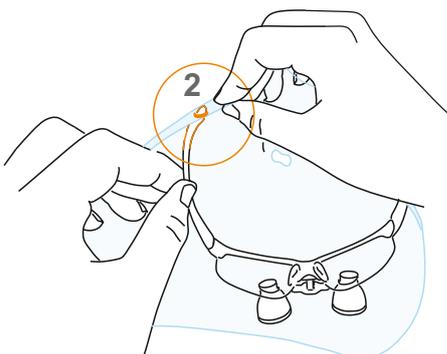5

7

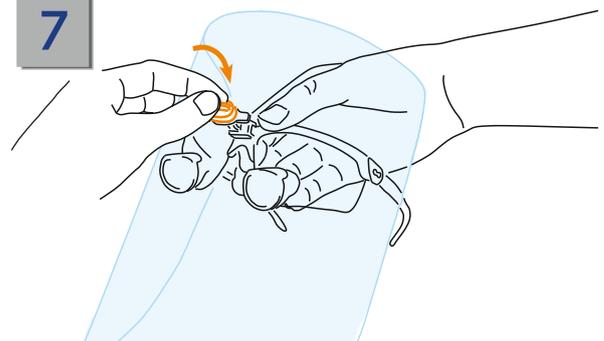Loch 2 einfädeln  
Insert into hole 2

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## CLEANING AND DISINFECTION

Cleaning should be carried out immediately after use of the **protect-on** to prevent blood or protein residues from solidifying.

1. Remove the magnetic cap, face protection foil and magnetic bar.  
Dispose face protection film properly.
2. Wipe off any residue from the magnetic cap and bar with a mild detergent; if necessary, remove residues of the detergent with damp, lint-free disposable cloths.  
  
Only use soft cloths for cleaning and disinfection and no sharp or scratching objects.
3. Wipe disinfection of the magnetic cap and bar with a cloth moistened with disinfectant. Allow the disinfectant to take effect as specified.

**Note:** Use the products listed below for disinfection.

4. Remove any residues of the disinfectant with sterile, damp, lint-free disposable cloths.
5. Dry thoroughly with a sterile, lint-free disposable cloth.

**CAUTION:** The face protection foils are intended for single use only and must not be reused.

The parts of the **protect-on** are **not** autoclavable. The resulting temperatures would destroy them.

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PRISMAN Pharma International AG; Am Stalden 16; CH-4622 Egerkingen
BODE Chemie GmbH; Melanchthonstraße 27; D-22525 Hamburg