Quickstart

spot-on[®]cordfree

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spot-on[®] cordfree


Factory Setting:

- The angle related on-off automatic switch is activated by raising the head/light up to the height at which the light turns off
- Luminous intensity: 10000 Lux


1.) Initial operation:

Insert the battery without touching the button (sensor is being calibrated).
The light turns on and off again after a few seconds and is ready now for use.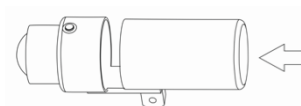

2.) Switching on and off:

Touch the button to switch the light on – the light turns off by touching once again. In case of difficulties with feeling the button, move your finger from the battery forward over the button.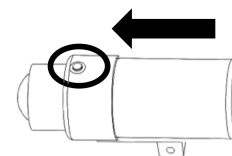


3.) Adjusting the brightness:

The adjustment mode can be activated by turning on the light and off again 3 times in a row. Each further contact adjusts the brightness from the lowest to the highest level and from the highest level back to the lowest. The latest intensity is saved after 3 seconds.


4.) Adjusting the angle related on-off automatic switch:

4 options: 3 different angles or permanent light.

a) Turn the light on and **b)** double tap with two fingers against the left or right temple within the first 15 seconds. The double tap is acknowledged by a short flash depending on whether it was successful (see picture on the right).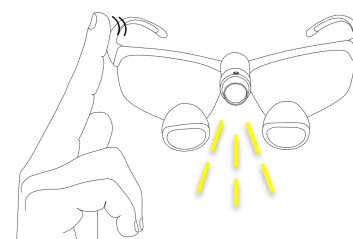

c) Turn the light on and off 3 times in a row to adjust the angle. 1x flash: **small** angle; 2x flash: **medium** angle; 3x flash: **large** angle. (see picture on the right).
4x flash (light flickers) angle is deactivated → permanent light.

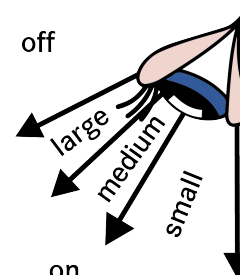The adjustment is saved after 6 seconds.

5.) Further information:

- Battery saving mode: the light turns off automatically if no further movement is recognized for more than 3 minutes after removing the loupes.
- RESET of the light can be performed by removing the battery (see 1.), however brightness and on-off automatic switch remain saved (unchanged).
- The battery is fully charged after approx. 1 hour.

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